

Your Days Rollator & Tri-Wheel Walker

Guide



days[®]
a Patterson Brand


PATTERSON
MEDICAL

Days Walkers...

...Keeping you on the move

Established in Bridgend, Wales in 1968, as a manufacturer of wheelchairs, Days Healthcare went on to become a market leader in the mobility market before being acquired by Patterson Medical in 2010.

With over 40 years of experience in providing reliable and well-engineered products to help people in their everyday life, the Days brand is trusted by consumers and healthcare professionals around the world.

Your Guide to Choosing a Walker


It is very important to have the frame at the correct height for use.

- If the frame is too high, you may find it difficult to straighten your elbows sufficiently and may not take enough body weight through your arms.
- If the frame is too low, it will encourage you to be bent over in a poor posture.
- When being measured for the height of your walking frame wear appropriate and supportive footwear.



Generally to ensure that the pushing handles are in the best position for weight bearing, the height of the handgrips should be at the level of the wrist bone when the user's elbows are very slightly bent (at an angle of about 15° flexion).

If purchasing a rollator with a seat, the appropriate seat height should be considered. When seated comfortably your feet should be firm and flat on the ground.

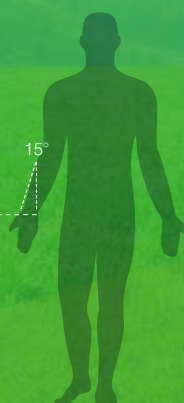

What's inside?

100 Series
pages 4 - 5

Deluxe Rollator
pages 6 - 7

110 & 111 Rollator
pages 8 - 9

2-in-1 Rollator
& Transit Chair
pages 10 - 11

Essentials 252 Rollator
pages 12 - 13

Lightweight Tri-Wheel Walkers
pages 14 - 15

Spares
pages 18 - 19

Steel Tri-Wheel Walkers
pages 16 - 17



Choosing a Rollator

Why Purchase a Rollator?

If a seat is required, then a rollator is the product for you.

- Offers more support than a tri-wheel walker
- Built-in rest seat and back support for comfort
- Different seat heights available on our range of rollators



Choosing a Tri-Wheel Walker

Why Purchase a Tri-Wheel Walker?

If you require support with your mobility, but you do not need a seat to take a rest, then the Tri-Wheel Walker will enable you to lead an independent life and keep you walking longer.

- The most manoeuvrable wheeled walking aid
- Easy folding for transportation and storage
- Lightweight

100 Series

Engineered to stringent standards

The 100 Series aluminium frame ensures a combination of lightweight design and robust build quality with a maximum user weight of 165kg (26st). The range is now available in **6 attractive colours**.



Maximum User Weight

3

Year Warranty

6

Colours Available

Ruby Red


Blue

Quartz

Racing Green

Purple

Russet Orange

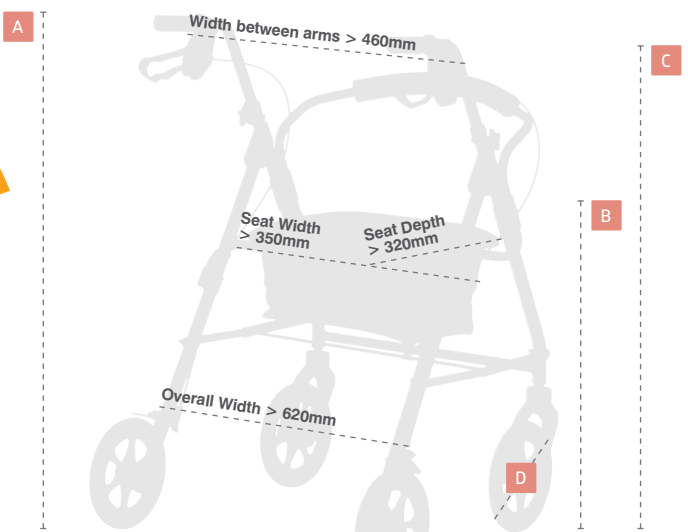

All models share the dimensions above

	102	103	105	106
	X-Small	Small	Medium	Large
	150 - 175cm 5'0 - 5'9"	155 - 180cm 5'1 - 5'11"	168 - 188cm 5'6 - 6'2"	175 - 200cm 5'9 - 6'5"
A	Handle Height 775 - 875mm 30½ - 34½"	Handle Height 780 - 900mm 30¾ - 35½"	Handle Height 840 - 940mm 33 - 37"	Handle Height 875 - 995mm 34½ - 39¼"
B	Seat Height 500mm / 19¾"	Seat Height 535mm / 21"	Seat Height 565mm / 22¼"	Seat Height 620mm / 24¼"
C	Backrest Height 750mm / 29½"	Backrest Height 800mm / 31½"	Backrest Height 800mm / 31½"	Backrest Height 870mm / 34 ¼"
D	Wheel Diameter 200mm / 8"	Wheel Diameter 200mm / 8"	Wheel Diameter 200mm / 8"	Wheel Diameter 200mm / 8"
	Product Weight 6kg	Product Weight 7kg	Product Weight 7.2kg	Product Weight 7.3kg
	Maximum User Weight			

Key Features

- Now with a 3 Year Warranty**
The robust 100 Series is made from tough materials to stand up to the rigours of everyday use. For added peace of mind the range is now supplied with a **3 Year Warranty**
- Available in 6 attractive colours**
- Ergonomic Handgrips & Adjustable Height Handles**
with 5 settings at 25mm intervals
- Loop Lockable Brakes** provide added security and convenience. The brake tension can also be easily adjusted to suit your requirements
- Available in four sizes** to accommodate a range of heights and builds
- Folds Easily for Storage or Transport**
Simply fold together by pulling the strap on the seat
- Comprehensive Spares Availability**
All wearable parts are easily replaced.
See pages 19-20

Ordering Information				
	102	103	105	106
Ruby Red	09 155 3049	09 155 3106	09 135 6328	09 155 3189
Blue	09 155 3023	09 155 3114	09 135 6336	09 135 6351
Quartz	09 155 3080	09 155 3098	09 155 3163	09 155 3197
Racing Green	09 135 6302	09 155 3148	09 155 3171	09 155 3221
Purple	09 155 3072	09 155 3130	09 154 6563	09 155 3213
Russet Orange	09 155 3056	09 155 3122	09 155 3155	09 155 3205

Deluxe Rollator

Sleek, modern and compact

The Days Deluxe Rollator is sleek, modern and folds neatly in a standing position for trouble free storage. It also folds into an incredibly compact size for easier transportation.



130 kg

20½ st

Maximum User Weight

1

Year Warranty

1

Colour Available

Silver

A technical line drawing of the Deluxe Rollator from a side profile. It includes dimension lines and labels: 'A' for handle height, 'B' for overall depth, 'C' for seat height, 'D' for seat depth, and 'E' for wheel diameter. Text annotations specify 'Seat Width > 460mm (18")' and 'Overall Width > 660mm (26")'.

A	Handle Height 810 - 930mm / (31¾ - 6½")
B	Overall Depth 680mm / (26¾")
C	Seat Height 610mm / (24")
D	Seat Depth 200mm / (7¾")
E	Wheel Diameter 195mm front / 165mm rear
Product Weight 7kg	
<div><div>130 kg</div><div>20½ st</div><div>Maximum User Weight</div></div>	

Key Features

Folds in a standing position, 28cm wide making it easy to store and at working height ready for use

Folds into a compact size, 66cm high, 28cm wide, 39cm deep for easier storage and transport

Walking stick holder supplied as standard

Adjustable Height Handles with 6 settings at 25mm intervals

Loop Lockable Brakes provide added security and convenience. The brake tension can also be easily adjusted to suit your requirements

Built in rest seat with canvas backrest

Comprehensive Spares Availability
All wearable parts are easily replaced.
See pages 19-20

Ordering Information	
Silver	09 143 9314

6

Deluxe Rollator

7

110 & 111 Rollators

Built for Strength

Our heavy-duty rollators are manufactured using heavy gauge steel tubing making them ideal for larger users.



110

178 kg

28 st

Maximum User Weight

111

317 kg

50 st

Maximum User Weight

1


Year Warranty

2

Colours Available


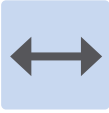



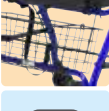


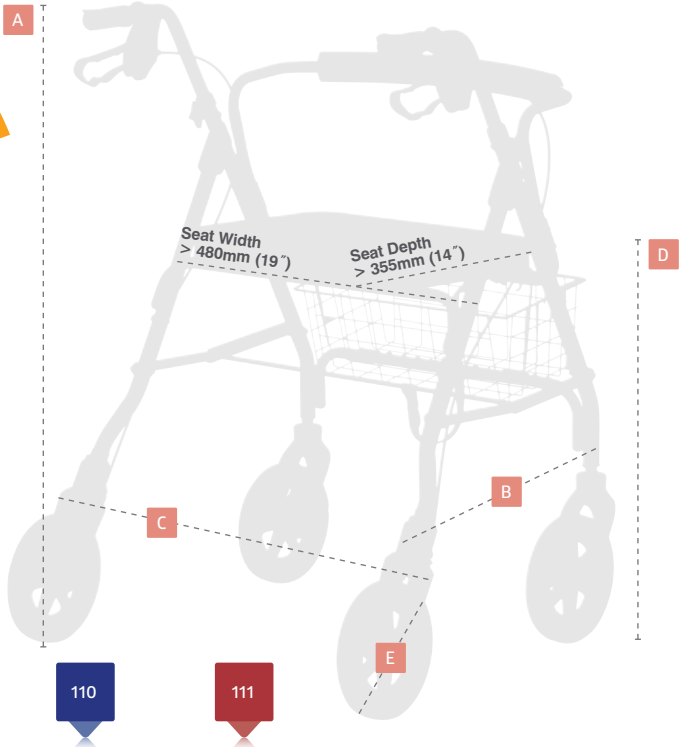110 Blue

111 Ruby Red



	110	111
A	Handle Height 830-940mm 32¾-37"	Handle Height 885-980mm 34¾-37½"
B	Overall Depth 725mm / 28½"	Overall Depth 660mm / 26"
C	Overall Width 670mm / 26½"	Overall Width 770mm / 30¼"
D	Seat Height 590mm / 23¼"	Seat Height 550mm / 21½"
E	Wheel Diameter 200mm / 8"	Wheel Diameter 200mm / 8"
	<div><div>178 kg</div><div>28 st</div><div>Maximum User Weight</div></div>	<div><div>317 kg</div><div>50 st</div><div>Maximum User Weight</div></div>
	Product Weight 12.3kg	Product Weight 13kg

Key Features

-  Available in 2 attractive colours
-  Wider Frames & Rest Seats for added comfort
-  High User Weights
-  Ergonomic Handgrips & Adjustable Height Handles with 5 settings at 25mm intervals
-  Loop Lockable Brakes provide added security and convenience. The brake tension can also be easily adjusted to suit your requirements
-  Under Seat Basket provides useful storage
-  Folds Easily for Storage or Transport Simply fold together by pulling the strap on the seat
-  Comprehensive Spares Availability All wearable parts are easily replaced. See pages 19-20

Ordering Information		
Blue	110	09 143 9090
Ruby Red	111	09 143 9124

2-in-1 Rollator & Transit Chair

Comfort When You Need It

This innovative product can be used as either a rollator or a short trip transport chair – the user can walk with the support of a rollator or can be pushed by a carer if they need a break.




160 kg
25 st
Maximum User Weight

1
Year Warranty

2
Colours Available

Silver

Ruby Red

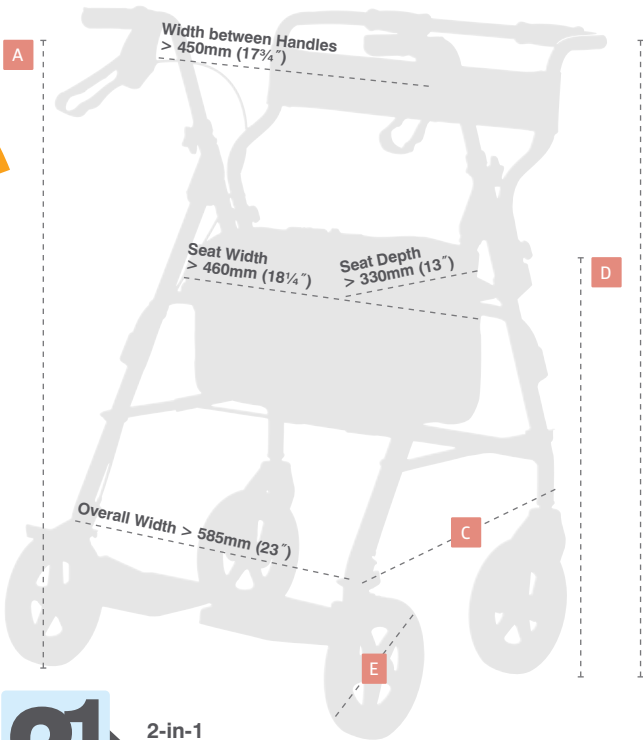

2-in-1

Suitable for use as a rollator or transit chair

A Rollator Handle Height
800-930mm / 31¼-36½"

B Pushing Handle Height
905mm / 35½"

C Overall Depth
740mm / 29¼"

D Seat Height
560mm / 22"

E Wheel Diameter
100mm / 8"

Product Weight
8.8kg

160 kg 25 st Maximum User Weight

Key Features



Available in 2 attractive colours



Comfort When You Need It

The seat and backrest are padded for comfort when seated



Flip up footrests

provide support during transport



Ergonomic Handgrips & Adjustable Height Handles
with 5 settings at 25mm intervals



Loop Lockable Brakes provide added security and convenience. The brake tension can also be easily adjusted to suit your requirements



Under seat bag

provides useful storage



Folds Easily for Storage or Transport

Simply fold together by pulling the strap on the seat



Comprehensive Spares Availability

All wearable parts are easily replaced.
See pages 19-20

Ordering Information

Silver	09 120 7299
Ruby Red	09 120 7307

Essentials 252 Rollator

Ideal for everyday use

Our entry level rollator is ideal for everyday use and provides excellent value for money. It has a lightweight frame, a more compact seat and a lower user weight than the 100 Series making it ideal for smaller users.



100 kg

15 3/4 st

Maximum User Weight

1

Year Warranty

1

Colour Available

Silver

A	Handle Height	795-905mm / 31 1/4-35 1/2"
B	Overall Depth	685mm / 27"
C	Seat Height	530mm / 20 3/4"
D	Backrest height	305mm / 12"
E	Wheel Diameter	200mm / 8"
Product Weight		6.42kg
		<div><div>100 kg</div><div>15 3/4 st</div><div>Maximum User Weight</div></div>

Key Features

- Ergonomic Handgrips** for optimum comfort
- Adjustable Height Handles** with 5 settings at 25mm intervals
- Loop Lockable Brakes** provide added security and convenience. The brake tension can also be easily adjusted to suit your requirements
- Comfort When You Need It**
The seat and backrest are padded for comfort when seated
- Folds Easily for Storage or Transport**
Simply fold together by pulling the strap on the seat
- Comprehensive Spares Availability**
All wearable parts are easily replaced. See pages 19-20

Ordering Information		
Silver	252	09 143 3713

12

Essentials 252 Rollator 13

Lightweight Tri-Wheel Walkers

Lightweight and easy to manoeuvre

The lightweight aluminium frame makes these walkers extremely easy to manoeuvre making them ideal for use indoors or out. The range is now available in **6 attractive colours** and supplied with a durable zipper bag as standard.



125 kg
19½ st
Maximum User Weight

1
Year Warranty

6
Colours Available

Ruby Red

Blue

Quartz

Graphite

Purple

Russet Orange

Overall Width
> 685mm / 27"

A Handle Height
830-930mm / 32¾"-36½"

B Overall Depth
685mm / 27"

C Wheel Diameter
190mm / 7½"

Product Weight
5kg

125 kg
19½ st
Maximum User Weight

Key Features



Available in 6 attractive colours



Ergonomic Handgrips
for optimum comfort



Adjustable Height Handles
with 5 settings at 25mm intervals



Loop Lockable Brakes provide added security and convenience. The brake tension can also be easily adjusted to suit your requirements



Durable Carry Bag
to transport and conceal your belongings



Folds Easily for Storage or Transport
using the simple easy to grab pull handle




Comprehensive Spares Availability
All wearable parts are easily replaced.
See pages 19-20

Ordering Information		
Ruby Red	240LR	09 155 3015
Blue	240L	09 135 6732
Quartz	240LQ	09 155 3007
Graphite	240LG	09 155 2728
Purple	240LPURPLE	09 154 1929
Russet Orange	240LRO	09 155 2710

Steel Tri-Wheel Walkers

Sturdy, robust and economical

Our range of Steel Tri-Wheel walkers are sturdy, robust and economical. Available in **4 attractive colours**.

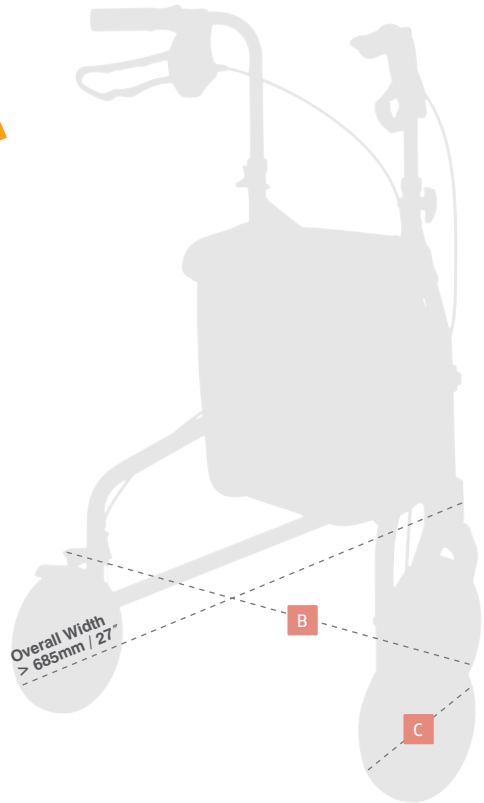

A Handle Height
830-930mm / 32¾"-36½"

B Overall Depth
610mm / 24"

C Wheel Diameter
190mm / 7½"

7.7 kg Weight of Frame

125 kg **19½ st** Maximum User Weight

Key Features



Available in 4 attractive colours



Ergonomic Handgrips
for optimum comfort



Adjustable Height Handles
with 5 settings at 25mm intervals



Loop Lockable Brakes provide added security and convenience. The brake tension can also be easily adjusted to suit your requirements



Optional Basket & Tray (240BT / 09 135 6724)

The optional basket and tray provides additional storage and a tray that can be used indoors for use when carrying your lunch from the kitchen



Folds Easily for Storage or Transport

Using the easy to grip handle that simply pulls the frame together



Comprehensive Spares Availability
All wearable parts are easily replaced.
See pages 19-20

Ordering Information

Chrome	240S	09 132 5505
Ruby Red	240SR	09 135 6765
Blue	240SB	09 135 6757
Grey	251	09 143 9355

125 kg **19½ st**
Maximum User Weight

1
Year Warranty

4
Colours Available

Chrome

Ruby Red

Blue

Grey

Spares

When you need them

100

Series*

page 4 - 5

- A** 09 147 6001 Right Handle & Brake Assembly
- B** 09 147 5995 Left Handle & Brake Assembly
- C** 09 147 6019 Height Adjustment Hand Knob
- D** 09 147 6027 Backrest
- E** 09 147 6035 Seat
- F** 09 147 6860 Seat clip
- G** 09 147 6068 Shopping Bag
- I** 102
- H** 09 147 6050 150mm/6" Front Wheel with Fork
- I** 103/105/106
- H** 09 147 6084 200mm/8" Front Wheel with Fork
- I** 102
- I** 09 147 6043 150mm/6" Rear Wheel
- I** 103/105/106
- I** 09 147 6076 200mm/8" Rear Wheel

Deluxe

Rollator

page 6 - 7

- A** 09 154 0020 Right Handle & Brake Assembly
- B** 09 154 0038 Left Handle & Brake Assembly
- C** 09 151 5998 Brake Cable Assembly
- D** 09 151 5972 Seat Upholstery
- E** 09 151 6038 Shopping Bag Support
- F** 09 151 6046 Shopping Bag 'mesh'
- G** 09 151 5931 Hand Knob
- H** 09 151 5949 Tipper
- I** 09 151 5964 Crutch Holder
- J** 09 151 6004 Brake Shoe - Left
- K** 09 151 6012 Brake Shoe - Right
- L** 09 151 6020 Cable Fixing
- M** 09 151 5915 Front Fork Carrier with Bolt
- M** 09 151 5923 Front Fork Assembly
- N** 09 151 5907 180mm / 7" Rear Wheel Assembly
- O** 09 151 5899 200mm / 8" Front Wheel Assembly

110

Rollator

page 8 - 9

- A** 09 146 2068 Right Hand Brake Assembly
- B** 09 146 2076 Left Hand Brake Assembly
- C** 09 146 2662 Height Adjustment Hand Knob
- D** 09 146 2050 Padded Backrest
- E** 09 146 2035 Seat
- F** 09 146 2043 Basket
- G** 09 146 2084 Front Wheel Assembly with Fork
- H** 09 146 2092 Rear Wheel Assembly

111

Rollator

page 8 - 9

- A** 09 146 2134 Right Hand Brake Assembly
- B** 09 146 2142 Left Hand Brake Assembly
- C** 09 146 2662 Height Adjustment Hand Knob
- D** 09 146 2126 Padded Backrest
- E** 09 146 2035 Seat
- F** 09 146 2118 Basket
- G** 09 146 2704 Front Wheel Assembly with Fork
- H** 09 147 5854 Rear Wheel Assembly

2-in-1 Rollator
& Transit Chair

page 10 - 11

- A** 09 153 9279 Right Handle & Brake Assembly
- B** 09 153 9287 Left Handle & Brake Assembly
- C** 09 147 5987 Brake Cable with Fittings
- D** 09 147 6019 Height Adjustment Hand Knob
- E** 09 154 5946 Transit Chair Footplate
- F** 09 154 0376 Front wheel with Fork & Fittings
- G** 09 154 0384 Rear Wheel with Fittings

Essentials

252 Rollator

page 12 - 13

- A** 09 154 6696 Right Handle Brake Assembly
- B** 09 154 6688 Left Handle Brake Assembly
- C** 09 147 5987 Brake Cable
- D** 09 135 1550 Height Adjustment Hand Knob
- E** 09 154 6704 Backrest
- F** 09 154 6712 Seat Pad with Fittings
- G** 09 154 6738 Basket
- H** 09 154 6720 Front Wheel with Fork
- I** 09 147 6043 Rear Wheel

Lightweight

Tri-Wheel Walkers*

page 14 - 15

- A** 09 153 3785 Right Handle & Brake Assembly
- A** 09 154 8064 Right Handle & Brake Assembly
(NEW Model batch code 2014-23)
- B** 09 153 3793 Left Handle & Brake Assembly
- B** 09 154 8072 Left Handle & Brake Assembly
(NEW Model batch code 2014-23)
- C** 09 147 5987 Brake Cable with Fittings
- D** 09 147 6019 Hand Knob & Fixings
- E** 09 135 6716 Shopping Bag
- F** 09 153 3819 Front Wheel with Fork & Fittings
- G** 09 153 3801 Rear Wheel with Fittings

Steel Tri-Wheel

Walkers*

page 16 - 17

- A** 09 146 2829 Right Handle Assembly
- B** 09 146 2837 Left Handle Assembly
- C** 09 118 1478 Brake Cable
- D** 09 135 1550 Hand Knob & Fixings
- E** 09 139 6712 Folding Link Assembly
- F** 09 139 7603 Double Tube Clamp
- G** 09 146 5756 Dome Nut
- H** 09 135 6716 Shopping Bag
- I** 09 147 6142 Front Wheel Assembly
- J** 09 147 6159 Rear Wheel Assembly



*Products purchased since January, 2011



Your Authorised Dealer:



Patterson Medical Ltd.

Nunn Brook Road, Huthwaite, Sutton-in-Ashfield, Nottinghamshire, NG17 2HU. UK

T: 08444 124 330

T: +44 1623 448 706

F: 08448 730 100

F: +44 1623 448 784

E: retail.sales@pattersonmedical.com

E: international@pattersonmedical.com

09 155 5267 1s1